



For information about Liquivet go to www.liquivet.com or email kdobson@myliquitab.com

©Liquivet™ and Liquitab® Systems Limited has patents or patents applications in a number of countries. Liquitab Systems Limited reserves the right to alter product specifications without notice as a result of our ongoing policy of product improvement.

© 2017 myLiquitab. All rights reserved. Made in China.

REV 11127

## Liquivet

Converts solid dose medications for animals, into an easy to administer flavoured liquid.

The Liquivet Unit is easy to use, compact, portable and robust.



www.liquivet.com

## Liquivet liquefies tablets so they're easy to administer.



## No messy crushing and mixing... it's as simple as 1,2,3.

- Place the pills and tablets in the Liquivet disposable beaker provided. Add water and the Liquivet flavouring of your choice. Open the lid of the unit and place the beaker inside.
- Close the lid of the Liquivet Unit, set the desired time and press start.
- The unit emits a 'beep' when the cycle is complete. Check the liquid to ensure that the tablets have been fully dispersed. If not repeat the cycle until all solids are dispersed in the liquid.

The Liquivet Unit is ideal for both veterinary and home use and our range of suitable flavourings, turns solid medications into a liquid treat that is easy to administer and that the animal will happily consume.

The Liquivet Unit is easy to use, compact, portable and robust.



Liquivet makes dispensing medications to animals easy. It features;

- A spill proof design
- Large, easy to use controls
- Portable and robust
- A variety of flavours to ensure the liquid is palatable for the animal

Liquivet saves time and is a safer way to administer animal medication.

